

Simplified, accurate testing for Flu A & B

Introducing our cost-effective, lateral flow flu rapid test supporting the reliable detection of Influenza A & B for appropriate treatment at the point of care.

Flu rapid test

Features and benefits

- CLIA-waived: no need for instrument
- Meets the influenza reclassification requirements for rapid antigen tests
- Fast and simple workflow: pre-filled vials, minimal hands-on time
- Results in 10 minutes
- Sample type: Nasal/NP swab
- Exceptional customer service: online training modules and technical support staffed by certified medical technologists/professionals

Precision

- **Sensitivity**
Flu A sensitivity: 90.3%
Flu B sensitivity: 88.0%
- **Specificity**
Flu A specificity: 96.7%
Flu B specificity: 99.2%

Convenient packaging

- 15-30°C/59-86°F storage requirement
- 24-month shelf life
- Ready-to-use: collection swabs and controls included




Procedure

- Twist** cap off the Extraction Buffer vial.
Note: Specimen must be extracted in the Extraction Buffer vial within 8 hours of collection.
- Insert** the swab through the ridges into the liquid.
Spin the swab 10 times to mix the specimen (while submerged).
Note: Nasal swabs may not reach the bottom of the vial. Ensure that the swab is fully immersed in the liquid when mixing.
- Discard** the swab.
Insert Test Stick (arrows pointing downward) into the vial.
Start timing.
- Read** test at 10 minutes. See Interpretation of Results.
Note: You may need to remove the Test Stick to read the test results.



IMPORTANT: Read and follow the complete test procedure before performing the test. Refer to the Instructions for Use (IFU) for more complete information. Users must follow manufacturer's instructions; failure to follow the instructions or modification to the test procedure will result in the test no longer meeting the requirements for waived classification.

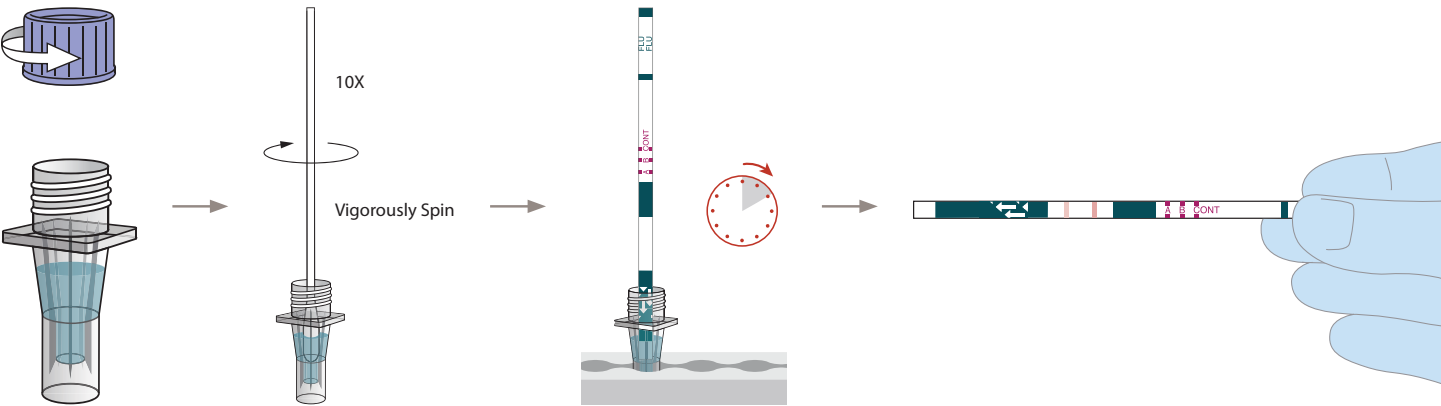

Interpretation of results

The Control Line must be present for the result to be valid. The appearance of ANY shade of a very light or faint pink to purple line at the A Test Line and/or B Test Line along with a C Control Line indicates a positive result for the presence of influenza A and/or B viral antigen. Even if you see a very light or faint pink to purple Test Line, as long as the Control Line is present, it is a positive test result.



Ordering information

Cat. No.	Description	Quantity
CH1032	Cardinal Health™ Flu A & B Test	25 tests per kit

For more information or to order, contact your Cardinal Health sales representative, call **800.964.5227** or visit **cardinalhealth.com/rapidtestkits**