

**Skin has many
stories to tell.
You just need to be
able to read them.**

HEINE dermatoscopes

Overview

It goes without saying that all of our dermatoscopes are premium products. But, still, they differ in one or two details. Read on for a summary of what these different details are and all their benefits:



	DELTA 30 PRO	DELTA 30	DELTAone	iC1	DELTA 20T	mini 3000
Polarisation	polarised + non-polarised (switchable)					non-polarised (immersion)
Illumination	LED ^{HQ} with colorSHIFT	LED ^{HQ}				
Free field of view – diameter in contact mode	30mm	30mm	21mm	digital	20mm	14mm
Magnification	10x	10x	10x	up to 40x digital	10x	10x
Handle options power sources	USB cable, table charger		USB cable		battery handle, USB rechargeable handle, rechargeable handle with table charger, EN 200 EN 200-1	battery handle, rechargeable handle with table charger
Contact plate	contact plate with scale				contact plate with scale, contact plate without scale	
Small contact plate*	optional			–	optional	
HEINE diSTANCE working ring	optional		–			
Digital documentation	with adapter			with iPhone**	with optional SLR photo adapter	–
Catalogue page	87	89	90	91	92	93

* for difficult-to-access pigmented lesions

**The Apple iPhone is not included in the scope of delivery. Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

HEINE DELTA[®] 30 PRO dermatoscope

High-end dermatoscope with our best imaging



Since the LED^{HQ} elements have been repositioned, the light reaches deeper layers of the skin too. This gives an extremely vivid, spatial impression of lesions – similar to a 3D image. The HEINE colorSHIFT function is a genuine world first. It allows you to easily adjust the colour temperature to suit your own viewing habits.

HEINE diSTANCE working ring: The stable working ring enables the user to examine and manage a lesion under magnification by maintaining a constant distance between the skin and the dermatoscope without contact. (optional)

EXTREMELY VIVID, SPATIAL IMPRESSION OF LESIONS

- ▶ Very good assessment of lesions' spatial structure possible due to deeper light penetration thanks to angled and directional LED^{HQ} arrangement
- ▶ Easier to distinguish between dark and white structures and patterns during polarisation
- ▶ Switch between polarised and non-polarised light at the touch of a button = toggle function
- ▶ Crystal-clear image, no matter whether you prefer to work close-up or further away
- ▶ Large field of view with 'real' 30 mm Ø due to 32 mm lens

CHOOSE THE LIGHT COLOUR TO SUIT YOUR OWN PREFERENCES WITH HEINE colorSHIFT

- ▶ 4 colour temperature stages from warm white to bluish white
- ▶ Adjustable to suit your own habits and preferences or the challenges posed by the lesion to be examined

STURDY AND DURABLE

- ▶ Particularly resistant to disinfectants due to new plastics
- ▶ Improved battery life

INTUITIVE AND ERGONOMIC

- ▶ Ergonomic handling due to angled product design
 - ▶ Scale on eyepiece shows the focus setting – for easy preselection of certain settings for specific users and/or for use with a camera or smartphone
 - ▶ Easy charging due to USB-C
 - ▶ Rechargeable battery with Li-ion technology
 - ▶ Fast change from contact to non-contact examination
 - ▶ Custom light intensity: 3 degrees of luminous intensity
 - ▶ 10x magnification
 - ▶ No cross-contamination: autoclavable contact plate with scale
 - ▶ Small contact plate for hard-to-reach lesions
- ▶ **Digital documentation:** Image and data management convenient and intuitive. More information can be found on our website: www.heine.com/en/digital



HEINE diSTANCE working ring



HEINE DELTA 30 PRO
film

HEINE DELTA® 30 PRO dermatoscope Sets



[01]

DELTA 30 PRO dermatoscope	LED
with contact plate with scale, E4-USBC power supply with cable, case [01]	K-235.28.305
with contact plate with scale, small contact plate, E4-USBC power supply with cable, charger 30 table charging station, universal smartphone adapter mag, case [02]	K-236.28.306
HEINE diSTANCE working ring [03]	K-000.34.103



[02]



[03]



► DIGITAL DOCUMENTATION: REMEMBER EACH AND EVERY MOLE

Image and data management convenient and intuitive. More information can be found on our website: www.heine.com/en/digital

► Accessories can be found on page 95.

HEINE DELTA® 30 dermatoscope

Reliable diagnostics



Digital documentation
by HEINE

Not only would the HEINE DELTA 30 win any beauty pageant, but stands out due to its very good imaging. Combined with a field of view measuring a veritable 30 millimetres and unprecedented colour rendering, in LED^{HQ} of course. Polarisation provides glare-free and non-reflective working conditions. Above all else, it is comfortable to operate with its angled, ergonomic design and intuitive control.

HEINE diSTANCE working ring: The stable working ring enables the user to examine and manage a lesion under magnification by maintaining a constant distance between the skin and the dermatoscope without contact. (optional)

- ▶ **Large true field of view of 30mm** (lens 32mm)
- ▶ **Crystal-clear image** with the achromatic HEINE optical system
- ▶ **Fast change from contact- to non-contact examination**
- ▶ **Ergonomic handling** thanks to the angled product design
- ▶ **Operate with just one hand:** individual focusing dioptre compensation approximately -4 to +4 dpt.
- ▶ **Excellent colour rendering** due to LED^{HQ}
- ▶ **Digital documentation:** Image and data management convenient and intuitive. More information on our website: www.heine.com/en/digital
- ▶ **Glare-free and non-reflective working conditions** thanks to polarisation
- ▶ **Custom light intensity:** 3 degrees of luminous intensity
- ▶ **10x magnification**
- ▶ **Switch between polarised and non-polarised light at the touch of a button** = toggle function. For more information, please visit heine.com/en/Toggle
- ▶ **No cross-contamination:** autoclavable contact plate with scale
- ▶ **Maintenance-free** due to our dustproof design
- ▶ **Rechargeable battery with Li-ion technology:** no memory effect; you can safely recharge the battery regardless of its charge state



[01]



[02]

DELTA 30 dermatoscope	LED
DELTA 30 with contact plate with scale, USB cord with medical approved plug-in power supply, case [01]	K-230.28.305
Charger 30 table charging station	X-002.99.212
HEINE diSTANCE working ring [02]	K-000.34.103

▶ **Accessories can be found on page 95.**

HEINE DELTAone dermatoscope

Uniting diagnostic precision and mobility



Small on the outside, big on the inside. This mobile marvel is taking medical professionals by storm. The crystal-clear image was accomplished using the achromatic HEINE optical system. What's more, it produces extraordinary colour rendering, due to the quality of our LED^{HQ}. Examinations are now particularly comfortable due to the polarisation allowing doctors to work glare-free and without disrupting reflections. These three features – clarity, LED^{HQ} and non-reflective working – plus the small dimensions, naturally, are what turn the DELTAone into a vital working tool wherever you go.



Digital documentation
by HEINE

- ▶ **Fits into any pocket**
- ▶ **Crystal-clear image** with the achromatic HEINE optical system
- ▶ **Fast change from contact to non-contact examination**
- ▶ **Excellent colour rendering** due to LED^{HQ}
- ▶ **Glare-free and non-reflective working conditions** thanks to polarisation
- ▶ **Operate with just one hand:** individual focusing dioptre compensation approximately -4 to +4 dpt.
- ▶ **Digital documentation:** Image and data management convenient and intuitive
More information on our website: www.heine.com/en/digital
- ▶ **Field of view of real 21 mm** (lens 22 mm)
- ▶ **Custom light intensity:** 3 degrees of luminous intensity
- ▶ **10x magnification**
- ▶ **Switch between polarised and non-polarised light at the touch of a button** = toggle function. For more information, please visit heine.com/en/Toggle
- ▶ **No cross-contamination:** autoclavable contact plate with scale
- ▶ **Maintenance-free** due to our dustproof design
- ▶ **Rechargeable battery with Li-ion technology:** no memory effect; you can safely recharge the battery regardless of its charge state



DELTAone dermatoscope	LED
with contact plate with scale, USB cord with medical approved plug-in power supply, case Standard Edition (black-silver)	K-210.28.305
with contact plate with scale, USB cord with medical approved plug-in power supply, case White Edition (white-silver)	K-211.28.305

▶ Accessories can be found on page 95.

HEINE iC1 dermatoscope

The digital dermatoscope



Digital documentation
by HEINE



- ▶ **Sharp, sharper, sharpest.** The HEINE iC1 is a device designed for capturing clinical and dermatoscopic images for documenting and supporting differential diagnosis of melanomas and other skin lesions. Developed in our unrivalled HEINE quality, the iC1 delivers sharp, shadow-free images.
Thanks, to both our precise, aspherical optical system and, the 12-megapixel image quality when using an iPhone* 6s or later model
- ▶ **Digital documentation:** Image and data management convenient and intuitive. More information on our website: www.heine.com/en/digital
- ▶ **Big, bigger, biggest.** The high-quality aspherical optical system with up to 40x true magnification offers everything you need in a practice
- ▶ **There's LED. And there's LED^{HQ}.** HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible. **That's what we call LED in HEINE Quality – or LED^{HQ}**
- ▶ **At the press of a button – the toggle function.** With two switching buttons, the option to switch back and forth between polarisation and immersion images is always within easy reach, regardless of the working position.
For more information, please visit heine.com/en/Toggle
- ▶ **Many see a smartphone. We see an exceptional camera and first-class display.** We've opted for the iPhone* for reasons of quality, user orientation and costs. From our perspective it offers the best camera, a display with the highest resolution and the most progressive touchscreen technology

iC1	LED
with contact plate with scale, mounting case smartphone iC1/5 for Apple iPhone 5s* and SE 1st generation*, USB cord with medical approved plug-in power supply	K-271.28.305
with contact plate with scale, mounting case smartphone iC1/6 for Apple iPhone 6/6s*, USB cord with medical approved plug-in power supply	K-270.28.305
with contact plate with scale, mounting case smartphone iC1/7 for Apple iPhone 7*, 8* and SE 2nd generation*, USB cord with medical approved plug-in power supply	K-273.28.305
with contact plate with scale, mounting case smartphone iC1/12 Pro for Apple iPhone 12 Pro*, USB cord with medical approved plug-in power supply	K-277.28.305

* The Apple iPhone is not included in the scope of delivery.
Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

HEINE DELTA® 20T dermatoscope

The classic



- ▶ **Crystal clear, high-resolution images** thanks to the high-quality achromatic optical system. The focusing system delivers 10 to 16x magnification for a detailed view of even the smallest structures
- ▶ **Toggle function for switching between polarised and non-polarised illumination mode.** With the press of a button. No exchange of contact plates necessary. The additional diagnostic function allows detection of “blink signs” when viewing crystalline structures and milia. For more information, please visit heine.com/en/Toggle
- ▶ **Glare-free and reflection-free view thanks to polarisation, even without immersion fluid.** For fast and flexible work
- ▶ **With just one hand:** individual focusing, range of correction approximately -6 to +6 dpt.
- ▶ **There's LED. And there's LED^{HQ}.** HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible. **That's what we call LED in HEINE Quality – or LED^{HQ}**
- ▶ **The DELTA 20T is also capable of digital documentation:** using the HEINE digital camera lens system, the dermatoscope can be connected to the SLR cameras of major brands without any issues. Specially developed photo adaptor lenses combine the camera and DELTA 20T to create an unbeatable pair



HEINE DELTA 20T
film



[01]

DELTA 20T dermatoscope	LED
with contact plate with scale, BETA4 USB rechargeable handle with USB cord and plug-in power supply [01]	K-010.28.388
with contact plate with scale, EN 200-1 wall transformer (single unit with one handle) as single module with long USB cord and E4-USB plug-in power supply [02]	K-010.28.252



[02]

- ▶ Accessories can be found on page 96.
- ▶ For information on HEINE battery and rechargeable handles and the EN 200 wall transformer, see pages 140-148.

HEINE mini 3000® LED dermatoscope

Compact pocket dermatoscope with LED^{HQ} illumination



Modern pocket dermatoscope with concentrated, bright light for uniform illumination with exact colour rendering. Unique, compact design with high quality optics. 10x magnification with focusing optics, metal instrument head. Can be used with the mini 3000 handle system. Handle available in black and blue, instrument head only available in black|silver.

- ▶ **Modern pocket dermatoscope with handy dimensions:** small on the outside, but big on the inside
- ▶ **10x magnification**
- ▶ **Focus setting for compensation of ametropia: ± 4 dpt.**
- ▶ **Instrument head made of metal**
- ▶ **20,000 ON|OFF switch cycles guaranteed**
- ▶ **Distortion-free image** with high resolution edge to edge thanks to high-quality, colour-neutral achromatic optical system
- ▶ **Special small contact plate** for the examination of difficult-to-access pigmented lesions, for instance between the fingers or toes
- ▶ **There's LED. And there's LED^{HQ}.** HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible. **That's what we call LED in HEINE Quality – or LED^{HQ}**
- ▶ **Maintenance-free LED^{HQ} illumination** – no longer a need to replace the light source
- ▶ **Exclusive battery performance indicator:** When the instrument is switched on, the maximum light intensity is shown, then the light intensity is regulated down to the level that corresponds to the current charge status of the battery so you know when your batteries need changing
- ▶ **Fade-out feature: brightness reduces slowly even with low residual capacity,** for even more efficient use of dry or rechargeable batteries

HEINE mini 3000® LED dermatoscope sets | kits



Set includes: mini 3000 LED dermatoscope,
with contact plate with or without scale, zipper case

	2.5V LED
mini 3000 battery handle (with batteries), with contact plate with scale	D-888.78.021



Kit includes: mini 3000 LED dermatoscope,
with contact plate with or without scale

	2.5V LED
mini 3000 battery handle (with batteries), with contact plate with scale	D-008.78.109
mini 3000 battery handle (with batteries), with contact plate without scale	D-008.78.107

Dermatoscopes

Further articles | accessories



[Dermatoscopes accessories](#)

While our instruments' quality starts with development, it certainly doesn't end with the accessories. Contact plates, smartphone adapters, digital camera adapters for DSLR and DSLM cameras, the HEINE diSTANCE working ring, the belt clip, cases, power supply units, chargers or USB cords.



[01]

DELTA 30 PRO | DELTA 30 and DELTAone

universal smartphone adapter mag [01]	K-000.34.273
---------------------------------------	--------------

You can find more information on the available iPhone* mounting cases, other products and accessories at www.heine.com/derm-accessories



[02]

[03]

DELTA 30 PRO | DELTA 30

contact plate with scale [02]	K-000.34.220
-------------------------------	--------------

small contact plate [03] for the examination of difficult-to-access pigmented lesions	K-000.34.206
------------------------------------------------------------------------------------------	--------------



[04]

[05]

HEINE diSTANCE working ring [04]	K-000.34.103
----------------------------------	--------------

charger 30 table charging station [05]	X-002.99.212
----------------------------------------	--------------



[06]

[07]

SLR SLM Camera Adapter universal [06]	K-000.34.140
-----------------------------------------	--------------

SLR SLM Camera Adapter flow M49 [07]	K-000.34.147
----------------------------------------	--------------

SLR SLM Camera Adapter flow M52	K-000.34.148
-----------------------------------	--------------

For other available camera adapters visit

www.heine.com/adapters

belt case [08]	K-230.01.000
----------------	--------------



[08]

DELTAone

contact plate with scale [09]	K-000.34.222
-------------------------------	--------------



[09]

[10]

small contact plate [10] for the examination of difficult-to-access pigmented lesions	K-000.24.207
------------------------------------------------------------------------------------------	--------------

belt case [11]	K-210.01.000
----------------	--------------



[11]

*Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries.

Dermatoscopes

Further articles | accessories

iC1 dermatoscope

contact plate with scale [01]	K-000.34.201
USB cord with E4-USB medical approved plug-in power supply	X-000.99.303



[01]

The currently available iPhone* mounting cases for iC1 can be found at

www.heine.com/en/mounting-case-smartphone

*Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries.

DELTA 20T

dermatoscope head with contact plate Ø 23 mm with scale, without handle [02]	K-008.34.221
dermatoscope head with contact plate Ø 23 mm without scale, without handle	K-008.34.222
contact plate 23 mm Ø with scale [03]	K-000.34.216
contact plate 23 mm Ø without scale	K-000.34.217
small contact plate, 8 mm Ø [04] for the examination of difficult-to-access pigmented lesions	K-000.34.205
SLR photo adaptor for Canon [05]	K-000.34.185
SLR photo adapter for Nikon	K-000.34.186
adapter cord (1.3m) for DELTA 20T handle	X-000.99.231
photo accessory set for Canon (includes SLR photo adapter, adapter cord, distance ring and BETA belt clip) [06]	K-000.34.190
photo accessory set for Nikon (includes SLR photo adapter, adapter cord, distance ring and BETA belt clip)	K-000.34.191
BETA belt clip for BETA handle	X-000.99.008



[02]



[03]

[04]



[05]



[06]

► For information on HEINE battery and rechargeable handles and the EN 200 wall transformer, see pages 140-148.

Dermatoscopes

Further articles | accessories



[01]

mini 3000 LED	2.5V LED
dermatoscope head* with contact plate with scale, without handle [01]	D-008.78.108

*only available in black|silver



[02]



[03]

contact plate with scale [02]	D-000.78.117
contact plate without scale	D-000.78.116
small contact plate, 8 mm Ø for the examination of difficult-to-access pigmented lesions [03]	K-000.34.008
zipper case [04]	D-887.00.000



[04]